



## Green state polisher | ZrO<sub>2</sub>

---



### Two-step green state ZrO<sub>2</sub> polisher.

Thanks to its dual working part and its special shape, this new “2 in 1” polisher is ideally suitable for work on pre-sintered zirconium oxide frameworks. The grey zone of the polisher is covered with a coarse polishing agent. It allows quick and controlled removal of stumps. The white tip of the working part is much finer. It is designed for retouching occlusal and interdental areas or crown margins. The polisher should be used at an optimum speed of 6.000 rpm to allow stress-free and precise guidance of the instrument. The zirconia blank benefits from the gentle, conservative function of the instrument. The risk of inadvertent damage to the chalk-like texture of the blank is greatly reduced and the smooth surfaces created by the polisher minimize the need for time-consuming retouching after sintering.

#### **Advantages in a nutshell:**

- one polisher for 2 polishing steps
- easy handling
- comfortable and precise work
- gentle on surfaces

### Appropriate use:

1. and 2. Removal of stumps with minimal vibration.
3. Smoothing of the surfaces - fast, safe and precise.



### Recommendations for use:

- For best results, use at an optimum speed of  $\text{rot.} \text{opt. } 6.000 \text{ rpm}$ .
- Work with low contact pressure.



9706.104.070